





# PocketVest™

## CIED Pocket Management System

### PocketVest Components

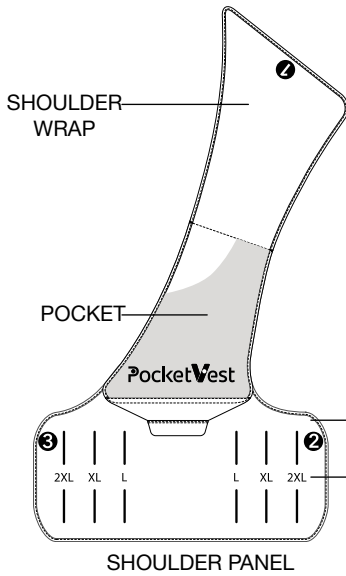


- † 6'1" OR TALLER WITH A LONG TORSO/SLEEVE LENGTH
- ‡ 5'8" - 6'0" TALL, WITH HIPS AND CHEST OF EQUAL CIRCUMFERENCE
- \* 5'3" - 5'7", WITH A SHORT TORSO OR SLEEVE LENGTH

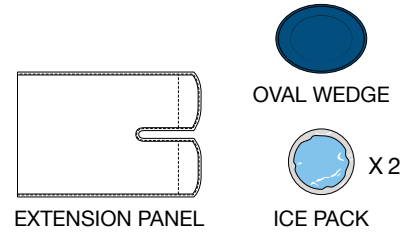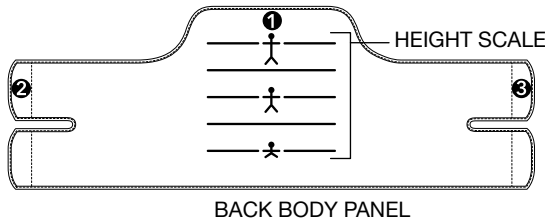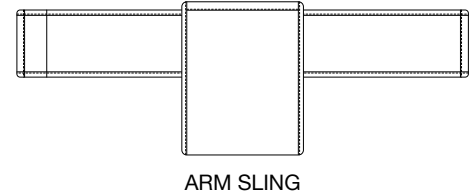


**STEP 1** DETERMINE PATIENT SIZING PER CHART BELOW. SET SHOULDER WRAP #1 AT PATIENT HEIGHT ON BACK BODY PANEL.


| PATIENT SIZING CHART |         |         |         |          |         |                |
|----------------------|---------|---------|---------|----------|---------|----------------|
| CHEST SIZE           | 30"-34" | 34"-38" | 38"-42" | 42"-46"  | 46"-50" | 50" OR GREATER |
| VEST SIZE            | XS      | S       | M       | L        | XL      | 2X (XXL)       |
| VEST PART #          | PV-01SM |         |         | PV-02LXL |         |                |

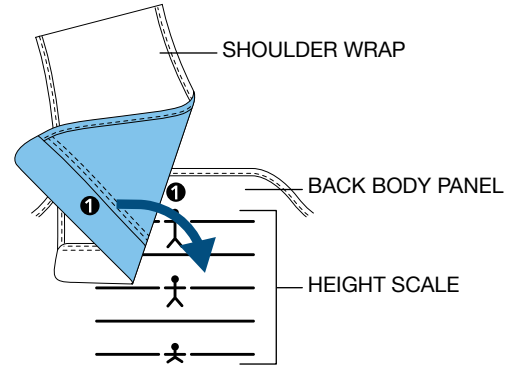

**STEP 2** ADJUST ABDOMINAL WRAP #2 ON FRONT BODY PANEL PER PATIENT SIZING CHART ABOVE.

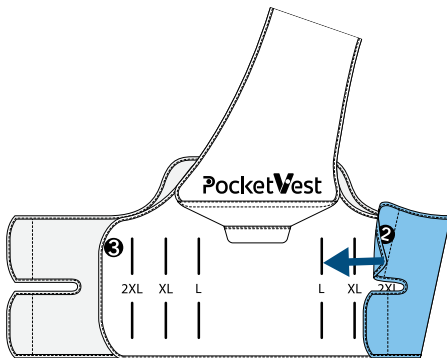


**STEP 3** GUIDE PATIENT'S LEFT ARM THRU VEST ARMHOLE. SECURE ABDOMINAL WRAP #3 ON FRONT BODY PANEL.

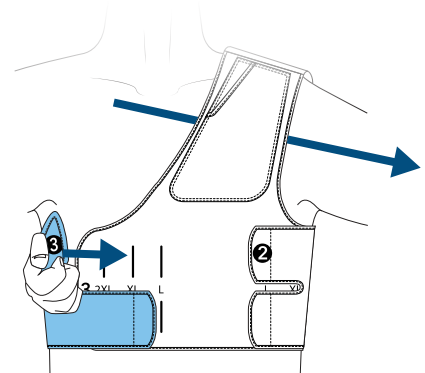

# PocketVest™


## CIED Pocket Management System

**STEP 4 OPEN SHOULDER POCKET (FIG 1). INSERT OVAL WEDGE WITH FOAM SIDE TOWARD SKIN AT INCISION SITE (FIG 2). SECURE DOWN POCKET COVER (FIG 3). OPTIONAL: PLACE GEL PACK SLIDE ICE UNDER OVAL WEDGE AND SLIDE IN SHOULDER POCKET (FIG 4).**

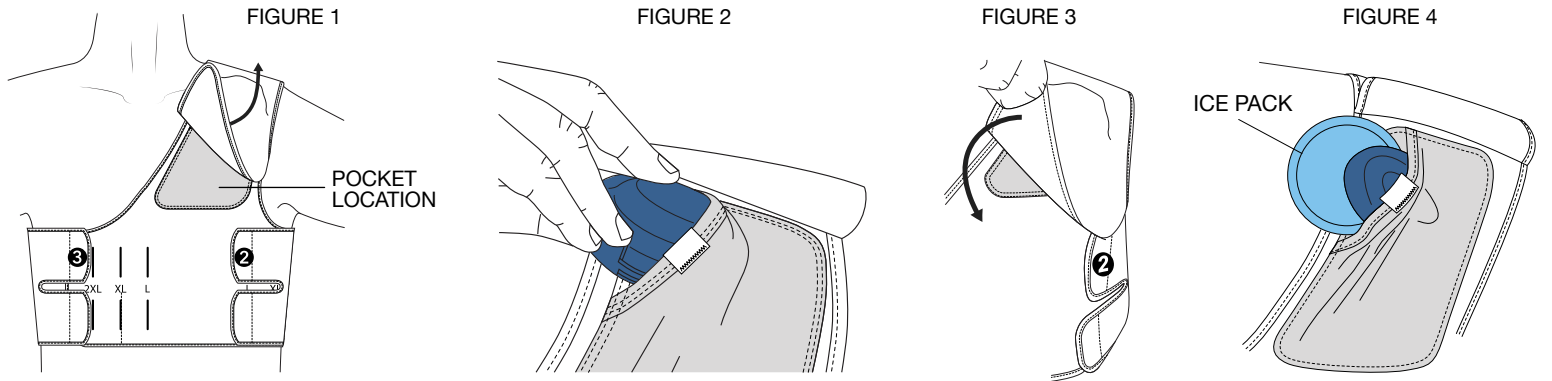


\*PLACE GEL PACK IN FREEZER AT LEAST 45 MINUTES BEFORE USE\*

**STEP 5 ATTACH ARM SLING STRAP TO ARM SLING ATTACHMENT ZONE ON BACK PANEL (FIG 1). SECURE SLING STRAP OVER LEFT ARM AND ATTACH TO FRONT BODY PANEL (FIG 2). GENTLY PLACE HAND IN ARM SLING AND SECURE (FIG 3).**

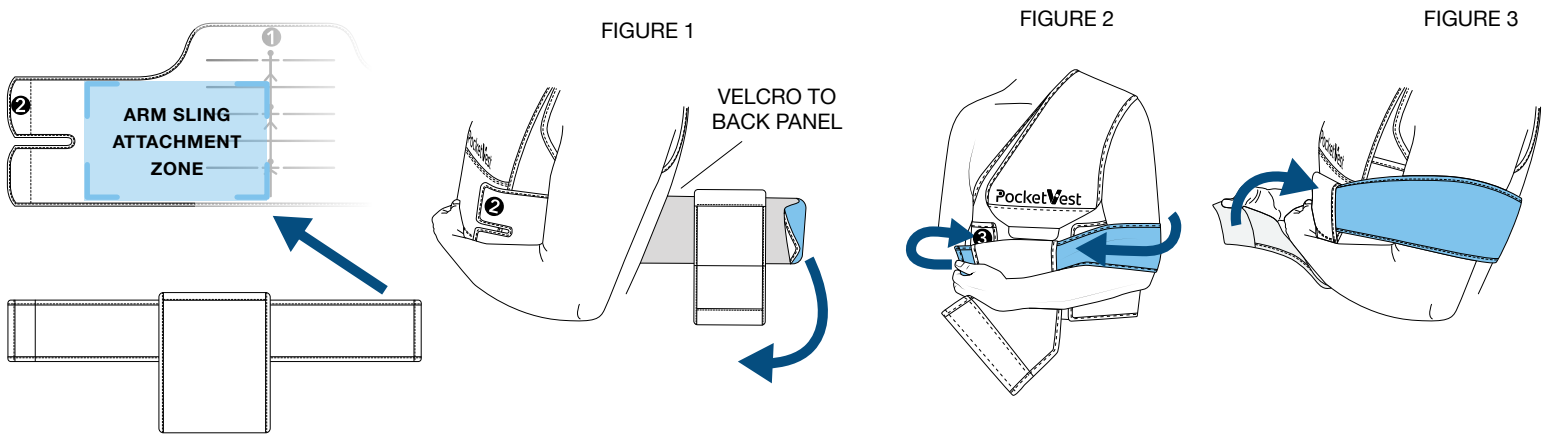

**STEP 6 OPTIONAL: ABDOMINAL EXTENSION PANEL FOR CHEST SIZE GREATER THAN 54". ATTACH EXTENSION PANEL TO BACK PANEL AND WRAP TO #3 ON FRONT BODY PANEL.**

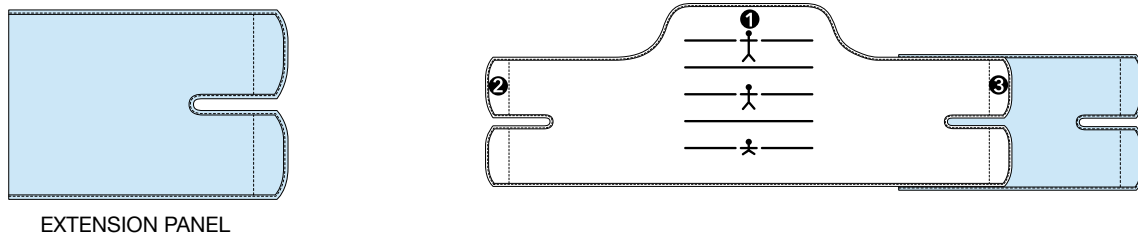

U.S. Patent 10,722,244. Other U.S. & Foreign patents pending.

**WARNINGS AND PRECAUTIONS:** For single patient use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.